

Instructions for Use Corona Virus (COVID-19) Antigen Rapid Test (Colloidal Gold)

For self-testing use

COMPONENTS



The Corona Virus (COVID-19) Antigen Rapid Test contains the following items to perform the test:

- 
1. Test card
 2. Instructions for use
 3. Extraction tube with buffer
 4. Sampling swab
 5. Tube rack
 6. Biohazard specimen bag

1. PREPARATIONS BEFORE TEST

- 
- 1 Read the Instructions for Use carefully.
- 
- 2 Clean and dry a flat surface in preparation for the test. Blow your nose with tissue and throw it away in a bin.
- 
- 3 Wash or sanitize your hands, and dry completely to avoid contaminating the test kit.
- 
- 4 Check the components to make sure nothing is damaged.
- 
- 5 Please scan the QR code to watch the operation video.

2. COLLECT AND PREPARE SAMPLE

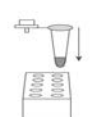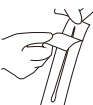- 
- 1 Remove the foil of the extraction tube and put it on the tube rack. (The tube rack can be found on the packaging for single-test)
- 
- 2 Take out the swab from its package and **DO NOT** touch the fabric soft tip of the swab.



HYTCG01S01(1 Test/Kit)
HYTCG01S05(5 Tests/Kit)
HYTCG01S30(30 Tests/Kit)


Please scan the QR code to access the operation video.
For assistance regarding the use of the product or for reporting any issues associated with the performance of the test, call 1300 999 668.
This service is available between 9 am and 7 pm (AEST), Monday-Sunday.




- 3 Carefully insert the entire soft end of the swab into one nostril, about up to 2.5 cm (1 inch) from the edge of the nostril. **DO NOT** insert the swab any deeper if you feel strong resistance or pain.




- 4 Slowly rotate the swab 5 times inside the nostril tissue. **Repeat this process with the same swab** for the other nostril to ensure that an adequate sample is collected from both nasal cavities.

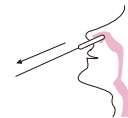

- 5 Slowly remove the swab from the nostril while rotating. Do not touch the soft tip, to avoid contamination.

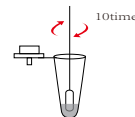

- 6 Insert the swab in the extraction tube and **swirl** the swab at least 10 times while pressing the swab against the bottom and side of the extraction tube.

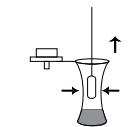

- 7 Gently pinch/squeeze the tube with your fingers, stir the swab head against the inside of the extraction tube. Pinch the extraction tube against swab, to release as much liquid as possible.




- 8 Insert a dropper tip into the extraction tube tightly.

1. Read the instructions for use carefully before test and follow the steps in order.
2. The test results will be displayed in 13-20 minutes. A clock/timer required, to measure the time. Do not read after 20 minutes.
3. Store the test kit at room temperature and dry place (2~30 °C), keep the kit away from direct sunlight.
4. Keep the test kit away from children.
5. Children and young people under the age of 18 should be supervised or assisted by an adult. This test is not intended for children under 2 years old.

3. TEST THE SAMPLE



- 1 Open the pouch and remove the test card. Lay it on the clean, flat surface.

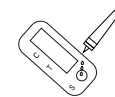


- 2 Add **four drops** of the solution into the well. Keep the tube vertical.




- 3 Immediately start timing and wait. Read test result at 13-20 minutes, **DO NOT** read the result after 20 minutes.


4. RESULTS INTERPRETATION

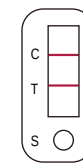

Positive result:
Both T line and C line appear, even faint lines.

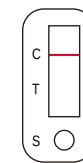

Negative result:
Only C line appears.

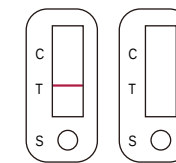

Invalid results:
C line doesn't appear.

Positive result

It indicates that antigens from SARS-CoV-2 (COVID-19) virus were detected, and you are very likely infected with the virus and presumed to be contagious. Staying at home protects the community. You should not visit high-risk settings such as aged and disability care. If you feel unwell or need COVID-19 advice for someone in your care, talk with your health provider, or refer to: your local state or territory health authority, or Australian Government health information.

Negative result

It may indicate that the result is negative or the patient has a low viral load that cannot be recognized by the test. A negative result may not mean the person is not infected. Repeat the test after 24-48 hours if you have COVID-19 like symptoms. If unwell, consults a medical practitioner for follow-up clinical care.

Invalid result


The control line may not appear due to insufficient sample volume or incorrect operation. An invalid result can be caused by a possibly incorrect test operation. Please repeat the test with a new test kit according to the instructions for use using a freshly collected sample. Do not interchange the components. If the second test result is invalid too but you are unwell, contact your healthcare provider for advice. Contact the Sponsor if more than one invalid result is obtained.

5. WASTE DISPOSAL

- 1 After the test completion, place the contaminated materials, such as test card, extraction tube and swab into the Biohazard specimen bag. Dispose of the sealed bag in closed household waste bins (not recycling) or according to your local regulations. **DO NOT REUSE.**



- 2 Wash or sanitize hands again after disposal.

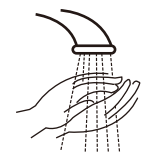

Epic Medical & Health Pty Ltd
Hotline 1300 999 668

Email: support@epicmedical.com.au
Operating hours Monday-Sunday
(9am - 7pm, AEST)