337, Avenue de Bruxelles

83507 La Seyne-sur-Mer

Cedex FRANCE

tél. 33 (0)4 94 10 98 10

fax (0)4 94 10 98 11

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Tel : 33 (0)4.94.10.98.10

Fax : 33 (0)4.94.10.98.11

PRODUCT INFORMATION FOR THE ATTENTION OF PATIENTS

HIGH COHESIVITY GEL PRE-FILLED BREAST IMPLANTS CE 0197

Your surgeon has considered you for a surgical procedure, and you have chosen to have a high cohesivity gel pre-filled breast prosthesis. PIP wants to inform you through your surgeon of the risks of such a procedure. The information in this document is designed so that you can better understand the advantages, as well as the risks of this type of procedure.

SURGICAL-RELATED RISKS

With this surgical procedure, the risk of a haematoma, infection or a necrosis is possible. Like with any wounding, it takes time and may lead to disturbances.

IMPLANT-RELATED RISKS

Implanting breast prostheses bears specific risks which can lead to revision surgery or possible removal of the prosthesis.

These specific risks are known and are still subject to scientific researches. They can be, for example :

- capsular contracture : this is a reaction to the foreign body that the implant represents, a fibrous shell appears around the implant which hardens progressively and compresses the implant. The hardening may be more or less prominent and may become unpleasant and distort the implant. A revision surgery may be necessary.

- Implant rupture : Since the envelope rupture favoured by the implant ageing is a complication that may not be seen, it is important to consult your surgeon in case of impact or strong pressure (fall, kick, accidents, etc...). Two types of rupture may occur : intracapsular rupture (small quantities of gel are kept within the periprosthetic capsule) and the extracapsular rupture. In the latter case, it is important to note the difference between the diffusion of small quantities of gel (nodule : silicone) or a significant rupture (silicone in the surrounding tissues).

- Asymmetry : certain factors such as capsu-

lar contracture, implant displacement after impact, or an unequal patient anatomy may result in asymmetry

- Possible loss of nipple sensitivity, partial or complete

- Folds : depending on the implant position, folds or creases may appear in post-surgery and / or in case of loss of weight

- Immunitary disorders : According to scientific studies, women with breast implants in general are not at an increased risk for autoimmune or connective tissue disease.

POST-OPERATIVE AWARENESS

Implanting breast implants implies meeting you surgeon's recommendations, especially :

- wearing a special bra day and night for 2 months in case of asymmetrical prostheses

- no baths before s/he allows you to

- no heavy loads to carry and no activities involving the pectoral muscles until s/he allows you (housecleaning, carrying children, sport...)

- a moderated and reasonable sun exposure

- pregnancy in the months following surgery is strongly to avoid

- during a mammography, the radiologist has to be informed of the presence of implants

- a regular medical follow-up with the necessity of informing your surgeon of any event related to your breasts

ALTERNATIVES

PIP has to remind you that other surgical procedures are existing nor not, alternative to the breast prosthesis implantation, especially in case of augmentation surgery.

- your surgeon can explain to you what is the mastopexy or the flap technique

- external prostheses or inflatable bras are also available

PRODUCT CONFORMITY

PIP breast implants are manufactured in

compliance with current Standards and Regulations, they are all CE-marked. PIP breast implants are guaranteed for 10 years, nevertheless PIP cannot be held responsible for any event related to badly appropriate living that might increase implant aging and/or lead to their rupture.

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